Helfende Hand

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Dear Customer,

Thank you for choosing our product and for the trust you have placed in us. In order for you and the user to enjoy your new product safely, conveniently, and comfortably, we kindly ask that you read the Instructions for Use. Doing this will also help avoid damage, injury, and incorrect operation and handling of the product. Please ensure that a copy of the Instructions for Use remains with the product so that every user has access to it.

If you have any questions, please contact your authorised distributor or us directly.

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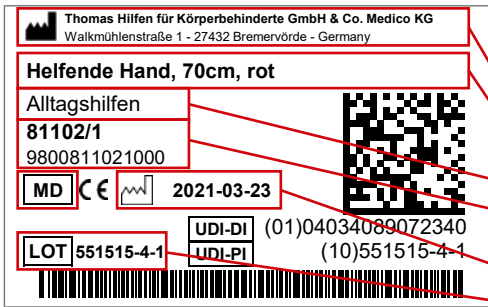
Phone: +49 (0) 4761 8860 · www.thomashilfen.de

We reserve the right to make technical changes. This can lead to differences between your product and how it is presented in brochures and other media.

Labels

Barcode Label

Position on the Product: Packaging

- Manufacturer
- Product Name
- Product Group
- Item Number short / long
- MD = Medical Device
- Manufacture Date
- Serial Number

Remarks

- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage and to be sure the product is in flawless condition.
- Keep the packaging material and small parts out of the reach of children to prevent swallowing or choking.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers.
- Clean the frame of the product as needed.
- Always wipe the product dry if it gets wet. Never put away wet after use.
- Do not transport fragile, burning or other dangerous objects.
- The maximum carrying capacity of the Hand Reacher is 3 kg / 6.6 lbs!
- If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.

Intended Purpose

Intended Use / Application Areas

The Hand Reacher is a class 1 medical device.

This aid for daily life can be used daily both indoors and outdoors. Users can singlehandedly grip and lift up objects that are farther away, enabling them to reach a little further. At the gripping end there is a magnet to lift metal objects more easily.

Any use that goes above and beyond this does not comply with the intended use.

Indication / Patient Target Group / Intended Users

The product is suitable for children, adolescents, and adults who have difficulty lifting objects due to functional injuries to the upper extremities and / or to the torso. Furthermore, it is suitable for people who have limited locomotor function because of joint disorders, neurological disorders, consequences of injuries, or deformities. The user has to have sufficient muscular strength in the hand and use of the hand.

Risks Associated with Use and Contraindications

If used properly, risks can be eliminated during use. There are no known contraindications.

Product Description

Your product is delivered with the standard equipment and comes preassembled. A copy of the Instructions for Use is enclosed with the product.

The user is taught how to use this device by trained staff who will reference these Instructions for Use.

Length and Colour Options

50 cm / 19.7"	60 cm / 23.6"	70 cm / 27.6"	80 cm / 31.5"	100 cm / 39.4"
yellow	green	red	blue	black

Adjustment Options / Features

To lift an object with the Hand Reacher, grasp it with the grasping claw by pulling the grasping lever to the handle with the grasping hand. Lift the object and move it to another place. If you now release the gripping lever again, the gripping claw opens and the object can be placed at the desired location.

For lifting small metal objects (e.g. keys), a magnet is integrated on the rear side of the handle.

Cleaning, Maintenance, and Repair

Cleaning, Care Instructions, and Disinfection on a Daily Basis

The plastic parts and metal parts on the product can be cleaned and disinfected with standard soap (neutral soap).

Clean the plastic parts, and metal parts before disinfecting. Damp wipe all the plastic and metal parts on the product with disinfectant. Only water-based products should be used. When disinfecting note the directions from the manufacturer in question.

Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry before storing the product.

Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at www.thomashilfen.de or requested from us directly.

Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

Period of Use

The product has a period of use of up to 2 years if used correctly and serviced regularly by a professional as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition. Time spent in storage at the distributor does not count towards the period of use.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor.

Service Life

The product has a service life of 2 years if the requirements stipulated for the period of use are met. The service life consists of the period of use and storage times.

Reuse

The product is not suitable for reuse.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG