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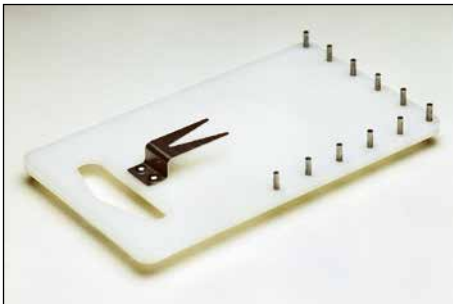
Dear Customer,

Thank you for choosing our product and for the trust you have placed in us. In order for you and the user to enjoy your new product safely, conveniently, and comfortably, we kindly ask that you read the Instructions for Use. Doing this will also help avoid damage, injury, and incorrect operation and handling of the product. Please ensure that a copy of the Instructions for Use remains with the product so that every user has access to it.

If you have any questions, please contact your authorised distributor or us directly.

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
Phone: +49 (0) 4761 8860 · www.thomashilfen.de

We reserve the right to make technical changes. This can lead to differences between your product and how it is presented in brochures and other media.

Label	Meaning
	<p>Barcode Label</p> <p>Position on the Product (Packing)</p> <ul style="list-style-type: none"> - Manufacturer - Product Name - Product Group - Item Number short / long - MD = Medical Device - Manufacture Date - Serial Number

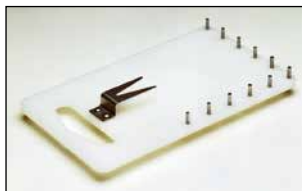9800800420000

9800803580000

9800800360000

Remarks

- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage and to be sure the product is in flawless condition.
- Keep the packaging material and small parts out of the reach of children to prevent swallowing or choking.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers.
- If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.

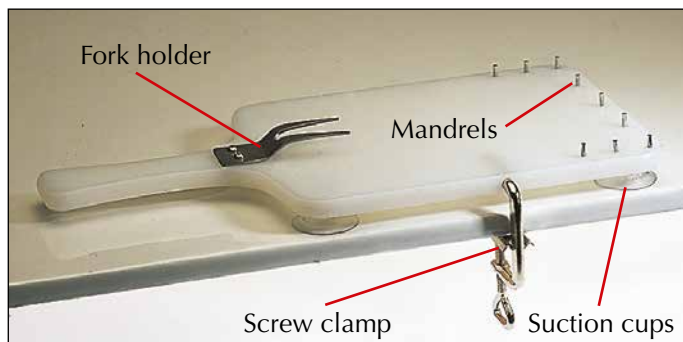
Product Description

Scope of Delivery

Your product is delivered with the standard equipment and comes preassembled. A copy of the Instructions for Use is enclosed with the product.
Only 9800800420000: The screw clamp is enclosed.

Standard Equipment

Plastic board; 4 suction cups; mandrels;
additionally at 9800800420000 and 9800803580000: fork holder;
additionally at 9800800420000: screw clamp

Teaching the User

The user is taught how to use this device by trained staff who will reference these Instructions for Use.

Adjustment Options / Features

The board is fixed in place by pressing firmly on the four suction cups to prevent it from slipping on a smooth surface.

The stainless steel spikes on the edge of the breakfast board / cutting board hold the bread and other food securely on the board.

Only 9800800420000 and 9800803580000: The firmly mounted fork can be used to fix sausage, cheese or the like so that they can also be cut with one hand.

Only 9800800420000: In addition, the breakfast board should - if possible - be fixed with the enclosed screw clamp. To do this, insert the metal pin into the hole on the side of the board and screw the board to the edge of the table.

Technical Data

Breakfast Board (Item 9800800420000)

Size (l x w)	38 x 18 cm / 15 x 7.1"
Weight	610 g / 1.3 lbs

Cutting Board (Item 9800803580000)

Size (l x w)	15 x 25 cm / 5.9 x 9.8"
Weight	291 g / 0.64 lbs

Cutting Board (Item 9800800360000)

Size (l x w)	16 x 16 cm / 6.3 x 6.3"
Weight	315 g / 0.69 lbs

Material Specifications

Plastic; Stainless steel

Cleaning and Maintenance

The product can be cleaned with standard dishwashing detergents or in standard dishwashers up to 60 °C.

Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry before storing the product.

Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

Period of Use

The product has a period of use of up to 6 years if used correctly and serviced regularly by a professional as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition. Time spent in storage at the distributor does not count towards the period of use.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor.

Service Life

The product has a service life of 8 years if the requirements stipulated for the period of use are met. The service life consists of the period of use and the storage times.

Reuse

The product is not suitable for reuse.

Only 9800800420000 Breakfast Board:

Intended Purpose

Intended Use / Application Areas

The breakfast board is a class 1 medical device.

The everyday aid enables the user to prepare and cut food independently. The positioning aids prevent the food from slipping off the board.

Any use that goes above and beyond this does not comply with the intended use.

Indication / Patient Target Group / Intended Users

The product is suitable for children, adolescents and adults with motor impairments in hand and / or fingers. Also suitable in case of one-handedness.

Risks Associated with Use and Contraindications

If used properly, risks can be eliminated during use. There are no known contraindications.

EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at www.thomashilfen.de or requested from us directly.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG